

Oral Flocked Swab

【Product Name】 Oral Flocked Swab

【Product Model】 612003

【specification】 Overall length:150mm

Flocked tip :20mm L

Tip Diameter: 5.0±0.2mm

Breakpoint : 80mm

Flocked tip, with ABS shaft. Sterile, individual bag pack



【Product structure】

The sampling swab consists of a sampling head and a sampling handle. The sampling head is a fluffy head made of nylon fibre material. The sample handle material is made of ABS material.

【Intended Use】

It is mainly used for clinical cell sampling and smear.

【Contraindication】 No

【Steps to use】

1. Open the package, gently insert the sample swab into the sampling site while holding the handle.
2. Gently rotate the sampling swab 3~5 turns and slowly remove it.
3. The extracted sample is placed in the sample collection tube, and then sealed to complete the sampling.









【Precaution】

1. Please check the packaging of this product carefully before use. If it is damaged, it is strictly forbidden to use.
2. This product is for one-time use only. It should be strictly in accordance with the aseptic operation specifications. Please destroy it immediately after use. It is forbidden to reuse. The disposal of wastes shall be carried out in accordance with national environmental protection laws and regulations.
3. Please read the instruction manual carefully, pay attention to the expiry date of the product before use. Do not use it after the deadline.

【Sterilization】 Sterile by irradiation

【Shelf Life】 2 years

【Definitions of Signs】

Symbol	Meaning	Symbol	Meaning
	In vitro diagnostic medical device		Do not reuse
	Batch code		Manufacturer
	Date of manufacture		Do not use if package is damaged
	Use by		Sterile by irradiation